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### Preparation of working solutions

#### Bilirubin solution

Dissolve the content of the vial with Reagent 1 in 4.00 ml of dist. water. Solution contains exactly  $\underline{a}$   $\mu\text{mol/l}$  of bilirubin. Stable for approx. 3 days at (+2 to +8) °C in dark.

#### Albumin solution

Dissolve the content of the vial with Reagent 2 in 8.00 ml of dist. water. Solution contains albumin 20 g/l. Stable for min. one week at (+2 to +8) °C.

#### Calibration solutions

Solution No.	Bilirubin solution $\underline{a}$ $\mu\text{mol/l}$ (ml)	Albumin solution (ml)	Resulting bilirubin concentration ( $\mu\text{mol/l}$ )
1	0.10	1.90	0.050 $\cdot$ $\underline{a}$
2	0.25	1.75	0.125 $\cdot$ $\underline{a}$
3	0.50	1.50	0.250 $\cdot$ $\underline{a}$
4	0.75	1.25	0.375 $\cdot$ $\underline{a}$
5	1.00	1.00	0.500 $\cdot$ $\underline{a}$

where  $\underline{a}$  is the bilirubin concentration given on the label of Reagent 1.

#### Health Protection

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Reagents of the kit are not classified as dangerous. At an accidental ingestion drink approx. 0.5 l of water. At an eye contact flush immediately and thoroughly with large quantity of water. At a skin contact wash the skin with soap and warm water. In all serious cases of health damage contact the physician.

#### Waste disposal

All tested samples should be treated as potentially infectious and with the contingent rest of the reagents should be liquidated in accordance with any other local and national regulations relating to the safe handling of such materials. Put packaging paper waste and rinsed containers to recycling.

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