# IMULATEST

## ASLO KONTROLA

(ASLO CON 1x 2)

Kat. č. 10010253

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### Literatúra

Spaun, J. et al., International standard for anti-streptolysin-O. Bull. WHO 24 (1961) 271-279.

Dátum poslednej revízie: 30. 1. 2015

Vyrába Erba Lachema s.r.o.  
Karásek 1d, 621 00 Brno, CZ  
e-mail: diagnostics@erbalachema.com, www.erbalachema.com

# IMULATEST

## ASLO CONTROL

Cat. No. 10010253

The kit is intended for immunoturbidimetric quantitative determination of Antistreptolysin O in human serum (1 × 2 ml).

Batch	xxxxxx
Expiration	mm/rrrr

### Certified values

Parametr	Concentration	Range	Unit
ASLO	000	000 - 000	IU/ml

Referential material, where the values are related to:  
International standard SSI (1st International Standard for Antistreptolysin O)

### Characteristics

ASLO control is a synthetic liquid control material, that contains human Antistreptolysin O. It is used for control during determination of Antistreptolysin O in human serum. It contains sodium azide (0,075%).

### Way of use and stability

The preparation is ready for use. It is treated as control according to the instruction included to the diagnostic set.

If the kit is stored correctly at (+2 to +8)°C, it is stable till the expiration date stated on the cover. After use the vials must be screwed tightly and stored at (+2 to +8)°C. Avoid contamination, pipette and decantation of calibrator back to the vial must be excluded.

### Precautions in use

The set is specified only for use in vitro. To be handled by entitled and professionally educated person. Serum was tested for presence of HBsAg and antibodies against HIV and HCV with negative results. In spite of the fact, the presence of some pathogenic materials can not be excluded. For this reason the content of the set has to be treated as potentially infectious material.

Reagents of the kit are not classified like dangerous, but contain in low concentration (<0.1%) sodium azide, very toxic substance, dangerous for the environment.

### First aid

If swallowed, a half liter of water should be drunk. In case of contact with skin or eyes, rinse them out with drinking water. The qualified medical aid has to be provided.

### Waste disposal

All tested samples should be treated as potentially infectious and with the contingent rest of the reagents should be liquidated in accordance with any other local and national regulations relating to the safe handling of such materials.

Put packaging paper waste and containers to recycling.

### References

Spaun, J. et al., International standard for anti-streptolysin-O. Bull. WHO 24 (1961) 271-279.

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