# XL MULTICAL

Cat. No.	Pack Name	Packaging (Content)
XSYS0034	XL MULTICAL	4 x 3 ml

EN

## INTENDED USE

XL Multical is a lyophilized calibrator based on human serum. The concentrations and activities are suitable for calibration of clinical chemistry assays on automated clinical chemistry analyzers.

## STORAGE AND HANDLING

Unreconstituted lyophilized calibrator is stable till the expiry date mentioned on the label when stored at 2–8 °C.

Stability of the reconstituted calibrator\*:

at 15–25 °C	8 hours
at 2–8 °C	2 days
at (-25) – (-15) °C	4 weeks (when frozen once)

### \*Exceptions:

Stability of Total Bilirubin in the reconstituted calibrator (when stored protected from light):

at 15–25 °C	6 hours
at 2–8 °C	1 day
at (-25) – (-15) °C	2 weeks (when frozen once)

Stability of Direct Bilirubin in the reconstituted calibrator (when stored protected from light):

at 15–25 °C	3 hours
at 2–8 °C	8 hours
at (-25) – (-15) °C	2 weeks (when frozen once)

Store calibrator tightly capped and protected from light when not in use. Store vials upright to avoid spills and leakage. Seal the vials tightly after use to prevent evaporation.

## RECONSTITUTION

1. Remove the screw cap and rubber stopper from the vial to be used. Avoid loss of lyophilizate.
2. Add exactly 3.0 ml of distilled/deionized water using a volumetric pipette.
3. Replace the stopper and cap for each vial and dissolve the contents on a roller mixer completely by occasional gentle swirling within 30 minutes. Avoid the formation of foam.

## VALUE ASSIGNMENT

The calibrator values were determined using the method mentioned in the enclosed value sheet. The calibrator values were obtained via single determination performed in different laboratories and the values assigned by a consensus of the results obtained by these laboratories.

## LIMITATIONS

Erroneous results can occur from reconstitution inaccuracies and technical errors associated with the assay procedure.

Do not use the reconstituted calibrator if there is visible evidence of microbial growth in the vial.

## WARNING

Human source material used in the manufacturing of this product has been tested by FDA - approved methods and found non reactive for hepatitis B, surface antigen (HbsAg), antibody to Hepatitis C (HCV) and antibody to HIV1/2.

The test procedures do not guarantee that all infectious agents will be detected. Because no test method can offer complete assurance that Hepatitis B virus, Hepatitis C virus and HIV 1/2 or other infectious agents are absent, the material should be handled as potentially infectious.

Lot No.: xxxxxxxx

Expiry: MM/YYYY

## ASSAYED VALUES USING XL SysPack REAGENT

Short name	Parameter	Methodology	Unit	Value	Unit	Value
ALB	Albumin	BCG	g/dl	0.00	g/l	0.0
ALP	Alkaline phosphatase	IFCC with AMP buffer	U/l	0	µkat/l	0.00
ALT/GPT	Alanine aminotransferase	IFCC without PDP	U/l	0.0	µkat/l	0.00
AMY	Amylase	CNPG3	U/l	0	µkat/l	0.00
AST/GOT	Aspartate aminotransferase	IFCC without PDP	U/l	0.0	µkat/l	0.00
BID	Bilirubin Direct	Diazo (Walter & Gerarde)	mg/dl	0.00	µmol/l	0.0
		DCA	mg/dl	0.00	µmol/l	0.0
BIT	Bilirubin Total	Diazo (Walter & Gerarde)	mg/dl	0.00	µmol/l	0.0
		DCA	mg/dl	0.00	µmol/l	0.0
CA	Calcium	Arsenaso III	mg/dl	0.0	mmol/l	0.00
CHE	Cholinesterase	Butyrylthiocholine	U/l	0	µkat/l	0.0
CHOL	Cholesterol	CHOD-PAP	mg/dl	0	mmol/l	0.00
CK	Creatine kinase NAC	DGKC	U/l	0	µkat/l	0.00
CK MB	Creatine kinase MB	Immunoinhibition	U/l	0	µkat/l	0.00
CL	Chloride	Mercuric Thiocyanate	mmol/l	0	mmol/l	0
		Jaffe 's kinetic	mg/dl	0.00	µmol/l	0
CREA	Creatinine	Enzymatic	mg/dl	0.00	µmol/l	0
		Glupa-C	U/l	0	µkat/l	0.00
GGT	Gamma-glutamyl transferase	GOD-POD	mg/dl	0	mmol/l	0.0
LDH	Lactate dehydrogenase	DGKC	U/l	0	µkat/l	0.00
LIP	Lipase	Enzymatic colorimetric test	U/l	0	µkat/l	0.00
MG	Magnesium	Xylidyl Blue	mg/dl	0.00	mmol/l	0.00
PHOS	Phosphorus	Ammonium Molybdate UV	mg/dl	0.00	mmol/l	0.00
TG	Triglycerides	GPO-Trinder	mg/dl	0	mmol/l	0.00
TP	Total Protein	Biuret	g/dl	0.00	g/l	0.0
UA	Uric Acid	Uricase-Trinder	mg/dl	0.00	µmol/l	0
UREA	Urea	Urease - GLDH, kinetic	mg/dl	0.0	mmol/l	0.0

**SYMBOLS:** The following symbols are used in the labelling of kits:

REF Catalogue No

CE Mark - Device comply with the Directive 98/79/EC

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