

# ERBA XL AutoWash AC/AL

## Acid and Alkali Wash solution



For *in vitro* diagnostic use only

Cat. No.:	Pack Name:	Packaging (Content):
XSYS0082	XL AUTOWASH AC/AL	R1 (AC): 5 x 44 ml, R2 (AL): 5 x 44 ml

### INTENDED USE:

ERBA XL-AutoWash AC/AL is a solution used with Erba XL Auto analyzers.

### PRINCIPLE:

The performance of any Auto analyzer depends on the cleanliness of the micro cuvettes. The use of serum or plasma or urine samples results in protein deposition. Over a period of time this may interfere with the assay resulting in erroneous results.

Reagent R1 contains HCl, H<sub>3</sub>PO<sub>4</sub>, detergent. pH: 1.20 +/- 0.5

Reagent R2 contains NaOH, detergent. pH: 13.1 +/- 0.5

### STABILITY AND STORAGE:

The reagent is stable till the expiry date mentioned on the label when stored at room temperature 15–30°C.

### PRECAUTIONS AND WARNING:

For *in vitro* diagnostic use only. To be handled by entitled and professionally educated person.

### DIRECTIONS FOR USE:

Ready to use. Please put defined position on XL Auto analyzers. Weekly washing recommended.

**R1 and R2:** Reagents are classified as corrosive on the basis of pH value.

Reagent **R1** contains HCl and H<sub>3</sub>PO<sub>4</sub>

Reagent **R2** contains NaOH



Danger

### Hazard statement:

H290 May be corrosive to metals.

H314 Causes severe skin burns and eye damage.

### Precautionary statement:

P264 Wash thoroughly after handling.

P280 Wear protective gloves/protective clothing/eye protection.

P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water / shower.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

### SYMBOLS:

The following symbols are used in the labelling of kits:



Catalogue No




Batch Code



Expiry Date  
(Last day of the month)



Manufactured by

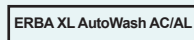

Product Name



CE Mark - Device comply with the Directive 98/79/EC



In Vitro Diagnostics



Consult Instruction for Use



Storage temperature



Content



Erba Lachema s.r.o.  
Karásek 1d, 621 00 Brno, CZ  
e-mail: [diagnostics@erbalachema.com](mailto:diagnostics@erbalachema.com)  
[www.erbamannheim.com](http://www.erbamannheim.com)

QUALITY SYSTEM CERTIFIED  
ISO 9001 ISO 13485