Suspensní médium pro ANAEROTest 23

Kat. č.: 10003367

Pro mikrobiologii

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Cat. No.: 10003367

For microbiology

Suspension medium for ANAEROTest 23 is supplementary preparation for MIKRO-LA-TEST® ANAEROTest 23 kit, which is intended for identification of anaerobic bacteria.

The Suspension medium for ANAEROTest 23 is supplied in glass tubes in the sterile conditions; the content of one tube (4 ml) is sufficient for preparing the suspension of one strain.

The preparation of the suspension of a bacterial strain should be carried out according to the procedure recommended for ANAEROTest 23.

2 packages (2x 20 glass tubes) serve for 1 package of ANAEROTest 23.

Storage, expiration:

The Suspension medium for ANAEROTest 23 should be stored at the temperature (+2 to +8) °C until the expiry date indicated on the package.

Warning:

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person.

Components of the kit are not classified as dangerous.

Waste disposal:

All tested samples should be treated as potentially infectious and with the contingent rest of the suspension medium should be liquidated in accordance with any other local and national regulations relating to the safe handling of such materials.

Put packaging paper waste and containers to recycling.

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