Suspenní médium MIC

Kat. č.: 10020337

Pro mikrobiologii

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Cat. No.: 10020337

For microbiology

Suspension Medium MIC is supplementary preparation for MIC kits (MIC G-I, G-II, URINE, NEFERM, STAPHY), which are designed for antibiotic susceptibility testing.

The Suspension Medium MIC is supplied in glass tubes in the sterile conditions; the content of one tube (13 ml) is sufficient for preparation of the suspension of one strain (inoculation of one MIC plate).

The preparation of the suspension should be carried out according to the procedure recommended for a particular type of MIC test.

Content:

30 tubes with suspension medium
Instructions for use

Storage, expiration:

the Suspension medium for MIC should be stored at the temperature of (2 – 25)°C until the expiry date indicated on the package.

Quality control:

Prepare suspension of control strain with density of McF 0.5. Transfer 60 µl of suspension into 13 ml of suspension medium MIC in tube. Incubate at 35±2°C for 16-20 hours.

Escherichia coli CCM 3954 (ATCC 25922) – turbid growth

Pseudomonas aeruginosa CCM 3955 (ATCC 27853) – turbid growth
Staphylococcus aureus CCM 4223 (ATCC 29213) – turbid growth

Warning:

For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Components of the kit are not classified as dangerous.

Waste disposal:

The residue of the suspension medium should be disposed in accordance with local and national regulations related to the safe handling of such materials. Recycle packaging paper waste.

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